

Stops External Bleeding From Wounds and Procedures



What is StatSeal Powder?

A topical powder that forms a seal around catheters to stop the flow of blood and exudate.

How Does It Work?


The mechanism of action is a simultaneous two-step action whereby the hydrophilic polymer rapidly dehydrates the blood and absorbs exudate while the potassium ferrate binds with the blood proteins to create a seal.

StatSeal Powder does not depend on the body's natural clotting mechanism to form a seal.

StatSeal Powder has been applied to the patient's access site. Do not be concerned by the color of the powder. The brown color is due to the powder ingredients.



**StatSeal Powder with PCD
(Powder Containment Device).**

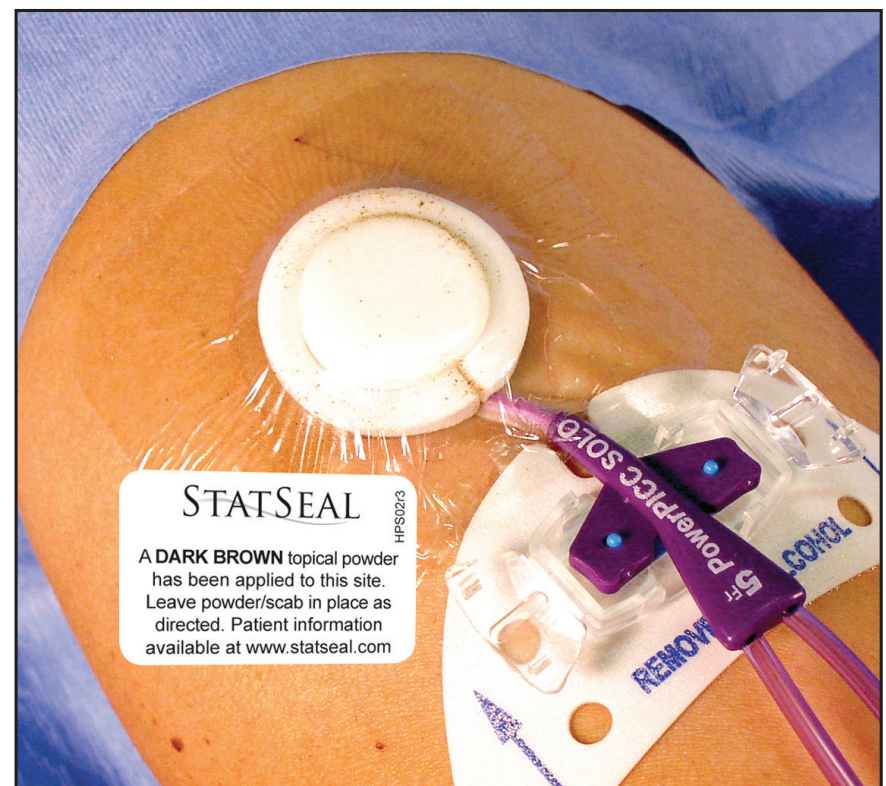

**StatSeal Powder with PCD
(Powder Containment Device) and PCD cover.**

If the patient experiences a rebleed:

- 1) Remove the dressing and reapply StatSeal Powder to the access site.
- 2) Apply firm pressure to the site on top of the powder for 2 minutes with dry gloved fingers or non-adherent dressing (do not use gauze) until hemostasis is achieved.