

Topical Powder That  
Stops External Bleeding  
from Wounds and Procedures



## Patient Discharge Instructions\*

StatSeal Powder has been applied to your access site.

### What is StatSeal Powder?



StatSeal is a non-prescription topical powder that:

- Stops bleeding
- Forms a protective seal over your wound or around your catheter.



### Caring for your wound or access site if you go home with your catheter

If you experience bleeding from the site, apply firm pressure to it. If bleeding continues, call your physician.

The seal should be left in place until it falls off naturally. **Do not disturb the StatSeal as this could cause rebleeding.**

### Call your physician if you notice:

- Bleeding and/or drainage at the site
- Increased pain at the site
- Increased swelling or redness
- Fever

Healthcare providers: Please visit [www.statseal.com](http://www.statseal.com) for more information on the powder and maintenance of the access site.

\* These instructions are also given to the patient upon discharge from the healthcare facility.

## Instrucciones para el uso del Polvo Hemostático Tópico, cuando el paciente ha sido dado de alta\*

El polvo StatSeal Powder ha sido aplicado en la zona intervenida.



### ¿Qué es StatSeal Powder?



StatSeal es un “polvo tópico “ que se expende sin prescripción facultativa y que:

- Detiene el sangrado
- Forma un sello protector sobre la herida o alrededor de su catéter.



### El cuidado de su herida o sitio de acceso si te vas a casa con el catéter

Si experimenta sangrado en la zoná, aplique presión sobre el área. Si el sangrado continúa consulte a su médico.

No remover el sello, déjelo caer naturalmente.

**No perturbar el StatSeal, ya que esto podría provocar un nuevo sangrado.**

### **Llame a su médico si nota que:**

- La zona está sangrando o está supurando
- Aumenta el dolor en la zona
- Aumenta la hinchazón o el enrojecimiento de la zona
- Fiebre

\* Estas instrucciones son exactamente iguales a las que se le proporcionan al paciente cuando es dado de alta de la institución de salud.