

STATSEAL DISC


Topical Disc
To Keep Access Sites
Dry And Intact



Patient Discharge Instructions*

StatSeal Disc has been applied to your access site.

What is StatSeal Disc?

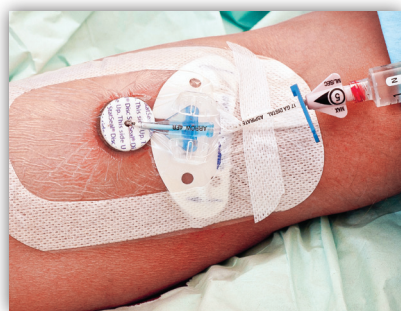

StatSeal Disc is a non-prescription topical disc made of compressed powder that:

- Seals your access site
- Stops bleeding
- Helps protect you against germs by forming a protective seal around your catheter.

Caring for your access site if you go home with your catheter

If you experience bleeding from the site, apply firm pressure to it. If bleeding continues, call your physician.

The disc should be left in place until your next dressing change. **Do not disturb the Seal as this could cause rebleeding.**

Call your physician if you notice:

- Bleeding and/or drainage at the access site
- Increased pain at the access site
- Increased swelling or redness
- Fever

Healthcare providers: Please visit www.statseal.com for more information on the disc and maintenance of the access site.

* These instructions are also given to the patient upon discharge from the healthcare facility.

STATSEAL DISC


Disco tópico
Para mantener los sitios de acceso
Secos e intactos,



Instrucciones para el uso de StatSeal Disco Hemostático Tópico, cuando el paciente ha sido dado de alta*

El disco polvo StatSeal ha sido aplicado en la zona intervenida.

¿Qué es StatSeal Disco?

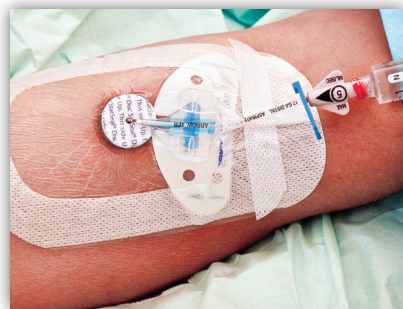

Disco StatSeal es un disco tópica sin receta hecha de polvo comprimido que:

- Sellos su sitio de acceso
- Detiene el sangrado
- Le ayuda a protegerse contra gérmenes formando un sello protector al rededor del cateter

Cómo cuidar la zona afectada si vas a casa con su catéter

Si experimenta sangrado en la zoná, aplique presión sobre el área. Si el sangrado continúa consulte a su médico.

No remover el disco su próximo cambio de vendaje.

No perturbar el sellar, ya que esto podría provocar un nuevo sangrado.

Llame a su médico si nota que:

- La zona está sangrando o está supurando
- Aumenta el dolor en la zona
- Aumenta la hinchazón o el enrojecimiento de la zona
- Fiebre

* Estas instrucciones son exactamente iguales a las que se le proporcionan al paciente cuando es dado de alta de la institución de salud.