

STATSEAL[®] DISC

Today's Technology to Keep
Access Sites Dry and Intact



Application includes 1 StatSeal[®] Disc.

StatSeal[®] Disc, when combined with manual pressure, forms a seal over a wound or procedural site.

Instructions for Use

Product Application:

1. Prepare the skin around insertion site according to facility protocol.
2. Open sterile packaging and remove disc.
3. Place the disc by sliding it down the catheter until it touches skin at the catheter site. The brown side should be towards the patient's skin and the slit perpendicular to the catheter. The edges of the slit should approximate one another to the extent possible. The foam side is facing up.

Tip: Don't stretch the disc apart too much upon application.

4. Apply a continuous hold on top of the foam for at least 2 minutes to form the seal; increase the hold time as clinically necessary.
5. Secure the line per protocol.
6. Cover with transparent dressing following facility protocol.
7. Remove disc with next dressing change.

Site should remain dry and intact until next scheduled dressing change.

Suggested Protocol for Dressing Change:

1. Stabilize the disc while removing transparent dressing with non-sterile glove.
2. Apply sterile gloves and working from the insertion site down.
 - a) Remove the StatSeal[®] disc using a gentle twisting motion.
 - b) Use sterile gauze to brush away any loose particles.
 - c) Remove the securement device.
 - d) Secure the catheter to the patient with tape or steri-strip to avoid catheter dislodgement.
4. Open the sterile field.
5. Apply a second set of sterile gloves.
6. Clean around the insertion site with hospital approved site prep, then allow site to fully dry. Do not cleanse to remove the seal.
7. After assessment of the insertion site:
 - a) Apply skin prep (once all loose powder is removed).
 - b) Secure the catheter
 - c) Apply the transparent dressing and label the site per facility protocol

Suggested Protocol for Line Discontinuation:

1. Remove the transparent dressing and securement device from the site.
2. Remove the disc and brush away any remaining product.
3. Remove line per protocol.
4. Dress site per facility protocol. Additional StatSeal[®] powder may be applied for bleeding as necessary.

